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## 1. Declaration of guarantee

E

Dear Customer,  
with this pressure infusion cuff you have chosen a quality product from ERKA. We are delighted that you have shown your confidence in ERKA. quality and we thank you. High quality materials, perfectly suited to their use give you the guarantee for optimal handling.

The cuff is made in accordance with the Directive 93/42 EEC dated June 14, 1993 for medical devices.

## 2. Description of instrument

The pressure infusion cuff consists of nylon fabric (100%polyamid).To maintain the correct shape a flat rod is sewn into the upper edge of the cuff. The white latticework fabric sewn on the upper side of the cuff consists of 100% polyester. The leather loop serves to hang the manometer. The rubber bag is inserted through the zipped opening in the rear of the cuff. The tubes are situated on the bottom of the cuff. The inflation bulb with reduction valve is fixed to one tube, the manometer to the other.

The reading scale of the manometer has a measuring range of 0-760 mmHg with a graduation of 100 mmHg. The scale is divided by colour-codes into two different measuring ranges:  
Green range: 0-300 mmHg Red range: 300-760 mmHg

## 3. Preparatory measures and carrying out

The transfusion-/infusion-bag is placed between the latticework fabric and the cuff. The two straps of different lengths on the upper edge serve to hang the cuff. The hook on the back of the manometer is hung on the leather loop.

The necessary pressure in the cuff is created by the inflation bulb. Take the bulb into your hand and check that the air release valve is closed (turn clockwise between thumb and index finger). The cuff is inflated by pumping the bulb. The pointer of the manometer goes up and shows the respective pressure in the cuff.

The necessary pressure is fixed according to the instructions of the responsible doctor.

After the transfusion/infusion is completed, open the air release valve (turn counter clockwise between thumb and index-finger) to vent the cuff completely.

## 4. Important hints

\*Avoid damaging the rubber parts with sharp or pointed objects

\*Protect the instrument from heavy vibration

\*Do not expose the instrument, in particular the rubber parts, to direct ultraviolet radiation

\*Protect the instrument from dirt and moisture

\*The cuff must only be inflated by the inflation bulb. Connection to an installed or mobile compressed-air-system is not permitted

\*Only original ERKA. parts are to be used for any spare or repair

## 5. Maintenance and care

### a) Manometer

For cleaning the manometer we recommend using a soft, dry cloth.

### b) Air release vale

Dust particles in the air can block the filter in the valve after a certain time. The bulb can be removed for cleaning or replacement. When changing the bulb make sure that the corresponding part of the main body and the inside of the bulb are free from grease to guarantee a tight connection.

### c) Cuff

#### Remove the rubber bag first

Washing: 30° mild washing + mild detergent

Wiping: with sponge or soft brush+mild detergent, rinse with water

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