Deutsch

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English

Check Control Inflator before use (see "TEST").

Use with VBM Laryngeal-Tube:

Before each use check both low-pressure cuffs for possible leakage and before insertion or removal completely deflate both cuffs with a syringe.

Connect the cuff pressure gauge with the connecting tubing to the inflation line for proximal and distal cuff. Afterwards inflate by pumping a few times so that the cuffs are overpressurized to 80 cmH₂O. The proximal cuff is filled first which stabilizes the tube. Once the proximal cuff has adjusted to the anatomy of the patient the distal cuff will be inflated automatically. Now press the red deflate valve to adjust to the desired pressure between 60 and 70 cmH₂O. (range "Laryngeal-Tube") The pressure in the cuffs can be checked constantly by connecting the cuff pressure gauge. In case of a pressure increase press the red release valve to adjust to the desired pressure between 60 and 70 cmH₂O.

Use with tracheal tubes with high-volume low-pressure cuff:

Before use check the cuff. Prior to intubation or extubation all air should be withdrawn from the cuff and the inflation line should be closed.

Connect the cuff pressure gauge with the connecting tube to the inflation line of the tube. Inflate the cuff to a pressure of 60 - 90 cmH₂O. This ensures that the cuff is in close contact with the tracheal wall. Release the pressure immediately by pressing the red release valve until the pointer reaches the green range "Tracheal-Tube". We recommend the intracuff pressure to be above 22 cmH₂O due to the risk of aspiration or pneumonia and below 32 cmH₂O due to the risk of ischemia of the tracheal mucous membrane (see medical discourse of J. Stauffer in Respiratory Care, July 1999). Due to the permanent connection to the tracheal tube the cuff pressure gauge works as a monitor and possible pressure increase can be read and released with the red release valve to the desired pressure.

Cleaning Instructions

If necessary the device can be cleaned with a soft and fluff-free cloth dampened in a mild soap-sud. The device must not be dipped into any liquid! For disinfection wipe the device with a commercially available disinfectant for surfaces on the base of alcohol. The device cannot be sterilized or cleaned mechanically.

Attention: Do not clean the glass (plastic material) with aggressive agents!

CAUTION: *Federal Law restricts this device to sale by, or on the order of, a physician. For USA and Canada only.*

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VBM Medizintechnik GmbH