Deutsch

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English

Instructions For Use

The Control Inflator should only be used with tracheal tubes with high volume low pressure cuffs.

Before use the Control Inflator needs to be checked as follows (see "TEST"):

- Close connecting piece with the finger
- Inflate with balloon to 120 cmH₂O; value must be constant for 2-3 sec, if the pointer drops off the device needs repair by VBM.

Before use check the cuff. Prior to intubation or extubation withdraw all air from the cuff and close the inflation line.

Connect the Control Inflator with the connecting tube to the inflation line of the tube.

With the "minimal leak technique" determine the necessary cuff pressure. Release air with the red release valve until the desired pressure is reached. Afterwards set the green pointer with the knob in the middle of the scale to this pressure value which serves as a reference.

Due to permanent connection to the tracheal tube the device works as a monitor and pressure increase can be read and corrected to the desired pressure by the red release valve.

Cleaning Instructions

If necessary the device can be cleaned with a soft and fluff-free cloth dampened in a mild soap-sud.

The device must not be dipped into any liquid!

For disinfection wipe the device with a commercially available disinfectant for surfaces on the base of alcohol.

The device cannot be sterilized or cleaned mechanically.

Attention: Do not clean the glass (plastic material) with aggressive agents!

CAUTION: Federal Law restricts this device to sale by, or on the order of, a physician. For USA and Canada only.

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VBM Medizintechnik GmbH