- Ladegriff 3,5 V für die Steckdose
- Charging Handle 3.5 V for Power Sockets
- Poignée rechargeable 3,5 V pour la prise de courant
- Manico ricaricabile 3,5 V per la toma de corrente
- Manco ricaricabile 3,5 V para a toma de corrente
- Cabo carregador 3,5 V para a tomada
- Перезаряжаемая от сети рукоятка 3,5 В

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• Users Manual

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• Mode d'emploi

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User's Manual

Charging Handle 3.5 V for Power Sockets

Dear Customers, thank you for choosing a KaWe product. Our products are known for their high quality and long life. This KaWe product meets the EC standards 93/42/EWG (standards for medical products).

Please read these instructions thoroughly and carefully before attempting to use this product and heed the given instructions.

Familiarize yourself fully with this product before use.

Utilization:

The charging handle 3.5 V is only to be used by authorized professional personnel. It is intended to be used as a fixing device for accessory parts such as otoscope, dermatoscope heads and Laryngoscope blades.

Purpose:

A satisfactory operation will be guaranteed with the use of short 3.5 V KaWe charging batteries. Screw off the screw cap at the lower end of the charging handle and insert the charging battery. The polarity is not important. The accumulator can be inserted on either side. Next, reattach the screw cap. Unscrew the charging part from the handle and connect the plug with the power socket. Before the first use, the charging handle must first be charged for 24 hours. A completely discharged charging handle must be charged for 16 hours.

Unsuited use / Contraindication of the products:

Any use other than that described here is not in accordance with the intended use of the unit. The manufacturer is not liable for any resulting damages. The user alone bears the risk.

Operation:

In order to avoid dirt accumulation and dust, always store the unit in its packaging. The outside of the unit can be cleaned with a damp, soft and lint-free cloth. To disinfect the instrument, a cloth dampened with alcohol can be used. Never bring the charging handles into contact with liquid and make sure that no liquid penetrates the housing! Since the charging handles are not designed for operations, simple cleaning and/or disinfection with a surface disinfectant with an alcoholic base is sufficient. Do not use any scrubbing cleaning agents! The units cannot be sterilized.

ATTENTION:

Completely discharge NiMH accumulators and then recharge them in order to avoid the lazy battery effect and the resulting loss of capacity!

Turn on the sliding switch by pushing it down and moving it to the left. The light intensity (maximum switch position) can easily be regulated with the rheostat. Turn off the sliding switch by moving it back to the right to its starting position.

The diagnostic devices such as otoscope and dermatoscope heads are to be slipped on the battery handle. A locking device ensures the correct fit of the heads. To unscrew the head, push down the knurled sleeve. In this way, the adjustment will be released and the diagnostic head can be removed. Laryngoscope charging handles 3.5 V are provided with the standardized interlock connection.

Cleaning:

In order to avoid dirt accumulation and dust, always store the unit in its packaging. The outside of the unit can be cleaned with a damp, soft and lint-free cloth. To disinfect the instrument, a cloth dampened with alcohol can be used. Never bring the charging handles into contact with liquid and make sure that no liquid penetrates the housing! Since the charging handles are not designed for operations, simple cleaning and/or disinfection with a surface disinfectant with an alcoholic base is sufficient. Do not use any scrubbing cleaning agents! The units cannot be sterilized.

Manufacturer: **KaWe**

Warning: Only use 3.5 V KaWe accumulators.

Limitation of reuse preparation:

Frequent cleaning only minimally influences this instrument. The working life of the instrument usually depends on the wear and tear and damage by use.

Instructions

Workstation: Remove dirt accumulation on surfaces with a disposable cloth.

Storage and transport:

No special requirements.

It is recommended that the instruments be cleaned immediately after their use.

Cleaning preparation:

Manual cleaning:

Wipe off the surfaces with a cloth dampened with alcohol. Remove impurities with running water.

Automatic cleaning: Not possible / required.

Disinfection:

The concentration and reaction times given by the manufacturer must be followed.

Maintenance: Defective parts must be removed and/or replaced.

Check and performance test:

Prior to the first start-up, carefully screw together the accumulator part and the attachment part of the handle.

Packaging:

Individual packaging: a standard polyethylene bag can be used. The bag must correspond to the size of the instrument so that it closes properly.

Sterilization: Not possible / required.

Storage: No special requirements.

Additional information: Exclusively use KaWe spare parts.

Contacting the manufacturer: Address or phone number of the dealer or dial the number (+49 7141 68188-0)

Further information, maintenance, storage:

When used and stored properly, the charging handle will serve you many years.

Technical data:

Classification: Protection class II
Primary circuit: 240 V ~ (50 – 60 Hz) EU-Version
120 V ~ (50 – 60 Hz) US-Version
Secondary circuit: 3,6 V
Charging current : 45 mA
First charging time: 24 hours
Charging time: 16 hours

Guarantee:

When used under normal circumstances and with attention to these instructions (the User's Manual) we guarantee this product for 2 years after purchase (except batteries). Should you need further information or should your instrument require repair, please contact your dealer!

Old electrical equipment:

Please dispose of your electric waste in accordance with legal requirements. Electric waste may not be disposed of together with domestic waste, and instead must be brought to a collection centre where it will be recycled. By complying with this regulation, you not only help protect the environment, but also the health of others. This regulation applies to the member states of the EU and other European countries with a separate accumulative system for electric equipment.
WEEE-Reg.-No. DE 63702395

Common accessories:

REF 01.73136.001 Otoscope head EUROLIGHT F.0.30, 3,5 V
REF 01.73232.021 Otoscope head COMBLIGHT F.0.30, 3,5 V
REF 01.91230.002 Dermatoscope head EUROLIGHT D30, 3,5 V
REF 12.80120.742 Charging battery, short, 3,5 V

Maintenance instructions for accumulators:

The charging batteries can be cleaned with a damp cloth and then be wiped off with a dry cloth. Please avoid short-circuiting the charging batteries.

Further maintenance instructions that may prolong the lifetime of the charging batteries:

Please heed the following instructions. The non-observance of these instructions may result in the accumulator losing its full capacity within a short period of time.

NiMH technology can stimulate a lazy battery effect that is analogous to the classic memory effect, which is triggered by the energy that has not been completely spent prior to charging the battery. This effect can be nearly eliminated by completely discharging the accumulator from time to time and then by recharging it two or three times in quick succession. Subsequently, the accumulator will regain its complete capacity.

In the case that the accumulator does no longer provide sufficient energy to operate the unit, immediately turn off the unit and charge the accumulator in order to prevent over-charging and the resulting destruction of the accumulator.

Please keep in mind that the charging batteries can spontaneously discharge. When the charging batteries are not used (charged) for an extended period of time, a self-discharge may occur. The capacity will be lost and the accumulators may become permanently defective. Therefore, when stored for more than one month, the accumulators should be partially charged to approx. 50%.

Please always respect the given charging time in order to prevent overcharging and the resulting destruction of the accumulator. NiMH-accumulators show a loss of capacity at low temperatures (starting at temperatures of 0°C).

• Keep the accumulator away from heating installations and open flames.

• The charging handle must not be dipped into water or salt water.

• Never try to disassemble or modify the accumulators or expose them to heat.

• Avoid dropping the accumulators and protect them against being hit or crushed as this may damage the housing.

• Please only use the accumulators and accessory parts recommended by KaWe.

The use of accumulators that are not explicitly approved for these units may cause explosions or leakages and the risk of fire, injuries and damages to surroundings. In the case that an accumulator obtains a leak and spilled substances come into contact with eyes, mouth, skin or clothing, immediately rinse the relevant sport with water and consult a doctor.

When charging is completed and the charging unit and the compact adapter are no longer in use, please separate them from the unit and the socket in order to avoid fires and other risks.

Do not cover the charging unit with any kind of material (table cloths, carpets, bedclothes, pillows etc.) during operation. If you use the adapter for an extended period of time, it may become overheated, lose its original shape or catch fire.

Please only charge the accumulator in the appropriate accumulator charging unit. The charging unit and the compact adapter have been developed especially for use with your product. Please do not use them with other products or accumulators.

All other products may cause the risk of excessive heat and distortion and thus the risk of fire and electric shock.

Prior to disposal of the accumulator, ensure that the contacts are masked with adhesive strips or other insulating material in order to prevent their direct contact with other materials.

Contact with other metallic material in trash cans may cause fire and explosions.

Disposal:

Please dispose of your rechargeable batteries in accordance with the legal regulations for **Ni Cd / NiMH**

Spare parts and accessories can be obtained in the KaWe price list or at our internet shop at **www.kawemed.de**.

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